<h1>RGP LENSES INSTRUCTIONS FOR USE</h1> <p><b>NOTICE D’UTILISATION DES LENTILLES RIGIDES (LRPG)</b></p>

<b>WWW.LABORATOIRE-LCS.COM</b>
14 place Gardin - 14000 Caen - France info@laboratoire-lcs.com

--

	<p>Consulter la notice - Read the package leaflet - Packungsbeilage lesen - Consulte las instrucciones de uso - Raadpleeg de gebruiksaawijzing - Consultar o folheto - Kilavuzza bkz - يجب الاطلاع على نشرة الاستعمال</p>
	<p>Référence commerciale de la lentille - Commercial lens reference - Bestellnummer der Linse - Referencia comercial de la lentilla - Referentie - Codice commerciale della lente - Referência comercial da lente - Kontakt linsen ticari referansi - المرجع التجاري للعدسة</p>
	<p>Date limite de 1<sup>er</sup> utilisation - Expiry date for first use - Haltbarkeitsdatum bis zur 1. Anwendung - Fecha límite del primer uso - Gebruiken voor - Data limite per il 1<sup>o</sup> utilizzo - Data limite da 1.<sup>o</sup> utilização - Son kullanim tarihi - التاريخ الأقصى لأول استعمال</p>
	<p>Numéro de série de la lentille - Lens serial number - Seriennummer der Linse - Número de serie de la lentilla - LOT nummer - Numero di serie della lente - Número de série da lente - Kontakt linsen seri numarasi - رقم العدسة المتسلسل</p>
	<p>Ne pas utiliser si conditionnement abîmé - Do not use if the packaging is damaged - Bei Beschädigung der Verpackung nicht mehr verwenden - Gebruik niet bij een beschadigde verpakking - No usar si el envase está dañado - Non utilizzare in caso di confezione danneggiato - Não utilizar se o material de acondicionamento estiver danificado - Ambalaj hasar görmüşse kullanmayın - عدم الاستعمال إذا كان التوضيب متضرراً</p>
	<p>Conformité aux exigences essentielles de santé et de sécurité et numéro de l’organisme notifié - Compliance with essential health requirements and notified body number - Einhaltung der grundlegenden Gesundheits- und - Sicherheitsanforderungen und Nummer der benannten Stelle - Overeenstemming met de essentiële veiligheids- en gezondheidseisen en het nummer van de aangemelde instantie - Conformidad con los requisitos esenciales de salud y seguridad y número del organismo notificado - Conformità ai requisiti essenziali per la salute e la sicurezza e numero dell’organismo notificato - Conformidade com as principais exigências de saúde e de segurança e número do organismo notificado - Sağlık ve güvenliği ilişkin temel gereklere uygunluk ve beyan edilen kuruluş numarasi - المطابقة مع متطلبات الصحة والسلامة الأساسية ورقم هيئة الاشعار</p>

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<b>ENGLISH</b>
<b>RGF LENSES INSTRUCTIONS FOR USE (RPG)</b>

This notice contains important information, read it, and keep it. Rigid Gaz Permeable (RGP) contact lenses are medical devices and should always be fitted by a contact lens specialist. For proper use, it is important that you follow all your specialist’s instructions and guidance on the packaging of your lenses and cleaning products.

- Your RGP lenses are wearable every day and need to be renewed every two years. (Follow your specialist’s recommendations).
- To avoid altering lenses material or geometry, it is essential to renew them according to the above modalities. Follow appointments planned by your specialist that are essential for controlling and following up your fitting.
- Wash your hands systematically before handling your lenses.
- RGP lenses are packaged in a mailer containing a Polyhexanide-based preservation solution. It is equipped with an inviolability system until the first opening. If the packaging is damaged, the lens should not be used.

#### Intended use

Rigid lenses are designed to correct the following ametropias: Myopia, Hyperopia, Astigmatism and Presbyopia. They are also indicated to compensate for certain corneal pathologies such as keratoconus and corneal irregularities.

Ophthalmologists or optometrists are the only professional people authorized to define rigid lenses type and parameters that should be prescribed according to your pathology and/or ametropia.

#### Lens insertion

- Make sure your lens is clean, in good condition and always immersed in the solution provided for this purpose. If the lens appears to be damaged, do not use it.
- Rinse your lens. Place the lens in the palm of your hand and rinse it with the contact lens solution recommended by your specialist to remove any impurities. It is strictly forbidden to rinse your contact lenses with tap water.
- Look at yourself in a mirror. Use your middle finger to spread the lower eyelid and your other hand to spread the upper eyelid.
- Approach the lens to your eye and gently place it on the cornea. Gently let go of the eyelids and blink several times to remove excess tears.
- Control your lens centering on your eye. If you notice air bubbles, a centering problem or discomfort, remove the lens and rest it using the same procedure. To improve comfort, you can put a drop of saline solution or lubricant in the lens before you put it down.

#### Lens removal

- Look in the mirror, position your index finger on the lower eyelid and the index finger of the other hand on the upper eyelid. With the index finger on the lower eyelid, apply a slight pressure and then raise the eyelid upwards to lift the lens. You can receive the lens with your thumb and index finger.
- Do not «pinch» the lens that is to be handled with care. Also be careful not to damage the lens with your nails.
- Using a plunger can make it easier to remove your lens.

#### Lens care

- Every day rub your lens for about 20 seconds with a rigid lens cleaning solution and then rinse it with a saline solution.
- Before decontaminating your lens, empty your lens case. Never reuse the solution that was left in the case.
- Place your lens in its case and fill it with a hard lens soaking solution following this solution instructions recommendations.
- After not using lenses for a week, decontamination solution in the case must be renewed before placing the lenses on the eyes.
- Read and follow the instructions of the contact lens solutions used.
- Follow your specialist’s recommendations.

#### Precautions for use

- Rigid lenses are meant to be worn during the day: put them on in the morning and remove them in the evening before going to bed
- Always make up after putting on your lenses and remove your make-up after you remove them.
- Do not use your saliva or any household products to maintain your lenses: only use the rigid lens care solutions recommended by your specialist.
- Do not change the brand of your contact lens solution without consulting your specialist.
- Do not put your lenses in contact with any cosmetic aerosol product.
- Pregnancy and some medications can disrupt tear secretion: talk to your specialist.
- Do not wear your lenses if you are exposed to dangerous or irritating fumes
- Lenses wearing is not recommended in case of eye pathology and in atmospheres of low humidity (airplane, helmet at the hairdresser…)
- Always carry eyeglasses (in case you must remove your lenses).

#### Contraindications and undesirable side effects

If you feel abnormal discomfort, remove your lens, and rinse it with a contact lens rinse solution.

Eye complications related to wearing lenses are rare if you follow the recommendations of professionals. However, if the following symptoms occur, remove your lenses, and contact your specialist:

- A red or irritated eye,
- A feeling of discomfort or persistent discomfort,
- An abnormal change in your vision,
- Increased sensitivity to light,
- Watery eyes or dryness feeling.

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