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<p>SCLERAL LENSES INSTRUCTION FOR USE</p>

- Your Scleral lenses are to be worn daily and should be replaced every two years at latest.

- Always wash your hands before handling your lenses.

Intended use

- Scleral lenses are designed for the following intended use:
 - Irregular corneas: Keratoconus, Pellucid Marginal degeneration, Corneal transplant with transparent but irregular cornea, Strong and/or irregular astigmatism, Corneal scars, High ametropia.
 - Alteration of corneal surface: Dry eye, Lyell or Stevens Johnson syndromes, Aftermath of eye burns, Terrien degeneration (marginal corneal ectasia), Chronic conjunctivitis with corneal epithelial destruction, Bubble Keratopathy.
 - RGP lenses or hybrid lenses failure
- Ophthalmologists or Optometrists are the only professionals who can define scleral lenses type and parameters suitable for pathology and/or correction to the patient.

Application

- Ensure that the lens is clean, in good condition and still immersed in the solution provided for this purpose. If the lens appears damaged, do not use it.
- Before applying for the first time, rub the lens during 20 seconds with a multipurpose solution.
- Place the lens in the palm of your hand and rinse with the solution recommended by your specialist, to eliminate any impurities. Contact lenses must not be rinsed with tap water under any circumstances whatsoever.
- Place your lens on the scleral suction cup and fully fill the lens with a preservative free saline solution.
- Look in a mirror that you have horizontally placed on a table. Use your middle finger to pull down the lower eyelid and your other hand to lift the upper eyelid.
- Bring the lens towards your eye and gently place it on the cornea and then squeeze the suction cup between the thumb and forefinger to release the lens. Gently let go off both eyelids and blink several times to clear excess of tears.
- Check that your lens is properly centred on your eye. If you notice any air bubbles, if it is off-centred, or if you experience discomfort, remove the lens and reapply according to the same procedure.

Removal

- Hold the pull suction cup between the thumb and forefinger.
- Look in the mirror in front of you, place your forefinger on the lower eyelid and the forefinger of the other hand on the upper eyelid.
- Stuck into the lower part of the lens and gently pull the lens up. The lens loses its adhesion, first from the bottom, then the entire lens will free itself.
- The lens should never “stick” to your eye. Consult your specialist if you have difficulty to remove the lens.

Lens care

- Each day, rub the lens for approximately 20 seconds with a cleaning solution for rigid lenses, then rinse using normal saline solution.
- Place your lens in the case and fulfil with the multipurpose solution, according to the instructions you will find in the solution’s leaflet. Do not reuse the residual solution.
- If the lenses have not been used for a week, the solution should be replaced before inserting your scleral lenses.
- Follow your contact lens specialist’s advice.

Precautions for use

- Always apply make-up after inserting your scleral lenses and remove make-up after taking them out.
- Do not use your saliva or any housekeeping products to clean your lenses: use only the products recommended by your specialist.
- Don’t change the brand of your cleaning product without consulting your specialist.
- Do not put your lenses in contact with hairspray or any other cosmetic spray product.
- Don’t sleep with your lenses
- Do not wear your lenses if you are exposed to dangerous or irritating fumes.
- Pregnancy and certain medications can disrupt tear secretion: talk about it to your specialist.

Contraindications and unwanted side effects

- If you feel abnormal discomfort, remove your lens and rinse it with a saline solution.
- Eye complications associated with wearing scleral lenses are rare if you follow professional’s advice. However, if the following symptoms occur, remove your contact lenses and contact your specialist:
 - Red or irritated eye
 - Feeling of discomfort or persistent problems
 - Abnormal change in your eyesight
 - Increased sensitivity to light

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