EYEBRID INSTRUCTIONS FOR USE

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WWW.LABORATOIRE-LCS.COM

14 place Gardin - 14000 Caen - France
info@laboratoire-lcs.com

⚠ Consulter la notice - Read the package leaflet -
Packungsbeilage lesen - Consulte las instrucciones de uso -
Raadpleeg de gebruiksaanwijzing - Consultar o folheto -
Kilavuza bkz - يجب الاطلاع على نشرة الاستعمال

REF

Référence commerciale de la lentille - Commercial lens reference - Bestellnummer der Linse - Referencia comercial de la lente - Referentie - Codice commerciale della lente - Referencia comercial da lente - Kontak lensicari referansi - المرجع التجاري للعدسة

SN

Date limite de 1^{re} utilisation - Expiry date for first use -
Halbarkeitsdatum bis zur 1. Anwendung - Fecha límite del primer uso - Gebruiksdatum voor - Data limite per il 1^o utilizzo - Data limite da 1.^a utilización - Son kullanım tarihi - التاریخ الاقصی لـ اول استعمال

CE
0000

Ne pas utiliser si conditionnement abîmé - Do not use if the packaging is damaged - Bei Beschädigung der Verpackung nicht mehr verwenden - Gebruik niet bij een beschadigde verpakking - No usar si el envase está dañado - Non utilizzare in caso di confezione danneggiato - Não utilizar se o material de acondicionamento estiver danificado - Ambalaj hasar görmüşü kullanmayın - عدم الاستعمال اذا كان التوصيب متضرراً

Conformité aux exigences essentielles de santé et de sécurité et numéro de l'organisme notifié - Compliance with essential health requirements and notified body number - Einhaltung der grundlegenden Gesundheits- und - Sicherheitsanforderungen und Nummer der benannten Stelle - Overeenstemming met de essentiële veiligheids- en gezondheidseisen in het nummer van de aangemelde instantie - Conformidad con los requisitos esenciales de salud y seguridad y número del organismo notificado - Conformità ai requisiti essenziali per la salute e la sicurezza e numero dell'organismo notificato - Conformidade com as principais exigências de saúde e de segurança e número do organismo notificado - Sağlıklı ve güvenilige ilişkin temel gereklere uygunluk ve beyan edilen kurulmuş numarası

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The EyeBrid Excel range benefits from a HydraPEG surface treatment. So be sure to use only contact lens solutions compatible with this HydraPEG treatment. Your specialist or optician can advise you on the contact lens solution compatible with those EyeBrid lenses.

Precautions for use

- EyeBrid lenses are meant to be worn during the day; put them on in the morning and remove them in the evening before going to bed.
- Always make up after putting on your lenses and remove your make-up after you remove them.
- Lens should never stay dry and should be kept away from heat sources.
- Do not use your saliva or household products for the maintenance of your lenses; use only the solutions recommended by your specialist.
- Do not change the brand of your soaking and cleaning solution without consulting your specialist.
- Do not put your lenses in contact with any cosmetic aerosol product.
- Do not keep your lenses to sleep.
- Pregnancy and some medications can disrupt tear secretion: talk to your specialist.
- EyeBrid Excel benefits from a treatment of surface HydraPEG. Así que asegúrese de usar solo soluciones de lentes de contacto compatibles con este tratamiento HydraPEG. Su especialista u óptico puede aconsejárselo sobre la solución de lentes de contacto compatible con esos lentes EyeBrid.

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ENGLISH

EYEBRID INSTRUCTIONS FOR USE

This notice contains important information, read it and keep it. EyeBrid lenses are medical devices and should always be fitted by a contact lens specialist. For proper use, it is important that you follow all your specialist's instructions and guidance on your lenses packaging and contact lens solution.

- Your EyeBrid lenses are wearable every day and need to be renewed every 6 months as they are part of the EyeBrid Excel range: EyeBrid (EyeBrid Silicone, EyeBrid Silicone Multifocal (ex), EyeBrid AirKone (ex), EyeBrid Oblate (ex)). (Follow your specialist's recommendations)
- To avoid altering the material or geometry of your lenses, it is essential to renew them according to the above modalities. Respect the appointments planned by your contact lens specialist that are essential to the control and follow-up of your fitting.
- Wash your hands systematically before handling your lenses.
- EyeBrid lenses are packaged in a mailer containing a Polyhexanide-based preservation solution. It is equipped with an inviolability system until the first opening. If the packaging is damaged, the lens should not be used.

Intended use

EyeBrid lenses are hybrid lenses with high oxygen permeability (high Dk) designed to correct the following ametropias: Myopia, Hyperopia, Astigmatism and Presbyopia. They are also indicated to compensate for certain corneal pathologies such as keratoconus and corneal irregularities.

Ophthalmologists or optometrists are the only professionals authorized to define EyeBrid type lenses and parameters that should be prescribed on the pathology and/or correction to be given to the patient.

Lens insertion

- Make sure your lens is clean and in good condition. If the lens appears to be damaged, do not use it.
- Rinse your lens. Place the lens in the palm of your hand and rinse it with the product recommended by your specialist to remove any impurities. It is strictly forbidden to rinse your contact lenses with tap water.
- Look at yourself in a mirror. Use your middle finger to spread the lower eyelid and your other hand to spread the upper eyelid.
- It is advisable to fill the EyeBrid lens with comfort drops or saline solution to improve comfort.
- Approach the lens to your eye and gently place it on the cornea. Gently let go of the eyelids and blink several times to remove excess tears.
- Control the centering of your lens on your eye. If you notice air bubbles, a centering problem or discomfort, remove the lens and insert it using the same procedure.

Lens removal

- Look in a mirror in front of you.
- Take a small suction cup to apply to the bottom of the EyeBrid lens (not in the center of the lens), i.e. on the soft part. Plunger should be 3/4 on the soft part, 1/4 on the rigid part.
- Pull the suction cup with a bottom motion up to take off the bottom skirt.
- Try again a few times if the lens does not peel off immediately.

Lens care

- Use a multipurpose contact lens solution for rigid or soft lenses daily. The lenses are also compatible with hydrogen peroxide or iodine povidone solutions. Be sure to use the lens cases dedicated to the use of those solutions and to follow the neutralization procedures.
- Read and follow the recommendations given in the instruction for use of the contact lens solution
- When removing the lens, gently rub your lens for about 20 seconds between the thumb and forefinger with a multipurpose contact lens solution, or, in case of thick tears or deposits, with a soft lens cleaning solution exclusively, then rinse the EyeBrid lens with a saline solution for contact lenses.

The EyeBrid Excel range benefits from a HydraPEG surface treatment. So be sure to use only contact lens solutions compatible with this HydraPEG treatment. Your specialist or optician can advise you on the contact lens solution compatible with those EyeBrid lenses.

Precautions for use

- EyeBrid lenses are meant to be worn during the day; put them on in the morning and remove them in the evening before going to bed.
- Always make up after putting on your lenses and remove your make-up after you remove them.
- Lens should never stay dry and should be kept away from heat sources.
- Do not use your saliva or household products for the maintenance of your lenses; use only the solutions recommended by your specialist.
- Do not change the brand of your soaking and cleaning solution without consulting your specialist.
- Do not put your lenses in contact with any cosmetic aerosol product.
- Do not keep your lenses to sleep.
- Pregnancy and some medications can disrupt tear secretion: talk to your specialist.

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Eye complications related to wearing lenses are rare if you follow the recommendations of professionals. However, if the following symptoms occur, remove your lenses, and contact your specialist:

- A red or irritated eye,
- A feeling of discomfort or persistent discomfort,
- An abnormal change in your vision,
- Increased sensitivity to light,
- Watery eyes or a feeling of dryness.

Contraindications and adverse effects

If you feel abnormal discomfort, remove your lens, and rinse it with a rinse solution.

Eye complications related to wearing lenses are rare if you follow the recommendations of professionals. However, if the following symptoms occur, remove your lenses, and contact your specialist:

- A red or irritated eye,
- A feeling of discomfort or persistent discomfort,
- An abnormal change in your vision,
- Increased sensitivity to light,
- Watery eyes or a feeling of dryness.

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