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ENGLISH

SOFT LENS INSTRUCTIONS FOR USE
--------------------------------

This is important information, read it and keep it.

Soft contact lenses are medical devices and should always be fitted by a contact lens specialist. For proper use, it is important that you follow all your specialist’s instructions and guidance on lenses packaging and cleaning solutions.

- Your soft lenses are wearable every day and need to be renewed quarterly, semi-annually, or annually depending on the type of lens (Follow your specialist’s recommendations)
- To avoid altering the material or geometry of your lenses, it is essential to renew them according to the defined periodicities. Follow the appointments planned by your specialist that are essential to the control and follow-up of your fitting.
- Wash your hands systematically before handling your lenses.
- Soft lenses are packaged in a sterile bottle containing a saline solution. The bottle is equipped with an inviolability system until the first opening. If the packaging is damaged, the lens should not be used.

#### Intended use

Soft lenses are intended to correct the following ametropias such as: Myopia, Hyperopia, Astigmatism and Presbyopia.

Ophthalmologists or optometrists are the only professionals authorized to define soft lenses type and parameters that should be prescribed based on your pathology and/or ametropia.

#### Lens insertion

- Soft lenses are delivered sterile and full packaging must be checked before the first use. Make sure the lens is clean and in good condition. If it looks damaged, do not use it.
- Rinse your lens. Place the lens in the palm of your hand and rinse it with the solution recommended by your specialist to remove any impurities. It is strictly forbidden to rinse your contact lenses with tap water.
- Look at yourself in a mirror. Use your middle finger to spread the lower eyelid and your other hand to spread the upper eyelid.
- Approach the lens to your eye and gently place it on the cornea. Gently let go of the eyelids and blink several times to remove excess tears.
- Control your lens centering on your eye. If you notice air bubbles, a centering problem or discomfort, remove the lens and rest it using the same procedure.

#### Lens removal

- Look at yourself in a mirror.
- Spread your eyelids with your fingers.
- Gently place your thumb and index finger on the lens that is on your eye. Pinch gently to remove the soft lens.

#### Lens care

- Everyday, rub your lens for about 20 seconds in the palm of your hand with a multipurpose solution for soft contact lenses, or, in case of greasy tears or deposits, with a soft lens cleaning solution exclusively, then rinse it with a saline solution.
- Before decontaminating your lens, empty your lens case. Never reuse the solution that was left in the case.
- Place your lens in its case and fill it with a multipurpose solution for soft lenses.
- Consider placing the right lens in the R (Right) and left compartment in the L (left) noted compartment.
- Placev a week without using lenses, the decontamination solution in the case must be renewed before placing the lenses on the eyes.
- Read and follow the instructions of the contact lens solutions used.
- Follow your specialist’s recommendations.

#### Precautions for use

- Soft lenses are meant to be worn during the day: put them on in the morning and remove them in the evening before going to bed.
- Always make up after putting on your lenses and remove your make-up after you remove them.
- Lens should never stay dry.
- Do not use your saliva or household products to maintain your lenses: only use the soft lens care solutions recommended by your specialist.
- Do not change the brand of your cleaning product without consulting your specialist.
- Do not put your lenses in contact with any cosmetic aerosol product.
- Do not keep your lenses to sleep.
- Pregnancy and some medications can disrupt tear secretion: talk to your specialist.
- Do not wear your lenses if you are exposed to dangerous or irritating fumes.
- Lenses wearing is not recommended in case of eye pathology and low humidity atmospheres (airplane, helmet at the hairdresser …)
- Always carry eyeglasses (in case you must remove your lenses).

#### Contraindications and adverse effects

If you feel abnormal discomfort, remove your lens, and rinse it with a contact lens rinse solution.

Eye complications related to wearing lenses are rare if you follow the recommendations of professionals. However, if the following symptoms occur, remove your lenses, and contact your specialist :

- A red or irritated eye,
- A feeling of discomfort or persistent discomfort,
- An abnormal change in your vision,
- Increased sensitivity to light,
- Watery eyes or a feeling of dryness.

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